Ferita[®]

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Ferita[®] Eilandpleister

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EN**General:**

FERITA® Eilandpleister is a primary wound dressing made of absorbent cotton pad laminated to a spunlaced non-woven with a medical grade, acrylic adhesive on a glassine release paper.

Indications:

FERITA® Eilandpleister may be applied to treating, binding up, fixing for various kinds of wounds, as well as the aseptic nursing of the wound after the operation, the acute injury wound, the small incision and scratch, and it also can be used for fixing the venous catheter.

Contra-indications:

We recommend that it should not be used on dry or covered with a hard black necrotic wound.

Appearance:

FERITA® Eilandpleister should be flat, regular cutting edge, but should not have defect like wrinkle, damage and obvious adhesive leakage, spill over ect.

Sterilization:

FERITA® Eilandpleister is sterilized by EO.

Packaging:

Each product is packed by poly paper pouch, and then put each to be packaged in one inner box.

Storage Condition:

FERITA® Eilandpleister should be stored in a well-ventilated clean room with relative humidity less than 80% and no corrosive gas.

The period of validity:

FERITA® Eilandpleister shall valid for 3 years from the sterilization date, provided that it is kept under the stipulated storage conditions.

Frequency of change:

The interval between dressing changes will depend entirely upon the state of the wound and the nature of the secondary dressing. We recommend that it should not be changed up to 3 days.

Method of use:

1. Make sure the wound area is clean and dry before applying the bandage applies.
2. A dressing should be chosen that covers the wound edge at least two centimeters overlaps.
3. The dressing should be placed with the smooth side against the skin and, if necessary, be held in place with tape or a bandage.
4. Before use, remove the protective film and stick the bandage on the wound area.
5. When the swollen bandage closes the edge after it has the discharge absorbed, the dressing should be replaced.

STERILE EO

The dressing have been sterilized with EO.

Do not use if package is damaged.

If package is broken do not use or re-sterilize product.

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