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Manufacturer:
Zhejiang Longterm Medical Technology Co., Ltd. Huancheng North Road 493,
Mo Gan Mountain National High-Tech District, 313200 Deqing, Zhejiang,
PEOPLE'S REPUBLIC OF CHINA

EU Representative:
Lotus Global Co., Ltd.
1 Four Seasons Terrace
West Drayton, Middlesex
London
UB7 9GG
UNITED KINGDOM

Importer:
White Label Pharma GmbH
Hohenzollernring 38-40
50672 Köln
GERMANY

Distributor:
meco Store GmbH
In den Auwiesen 6
69168 Wiesloch
GERMANY

General:

meco® Superabsorber is a sterile superabsorbent dressing for use on moderate to heavily exuding wounds.

Indications:

Exuding wounds such as venous leg ulcers, pressure sores, diabetic foot ulcers and other wounds producing moderate to high levels of exudate.

Product properties:

meco® Superabsorber absorbs and retains exudate in the dressing. The product wicks exudate vertically from either side, retaining it and forming a gel within the dressing. This minimizes the risk of maceration at the wound edges, prevents damage to the skin around the wound, and provides a moist wound environment.

Instructions for use:

meco® Superabsorber should be used on moderate to heavily exuding wounds where there is a need for high absorption and fewer dressing changes. meco® Superabsorber can be used both as a primary dressing and as a secondary dressing. Choose the size of the dressing according to the size of the wound to be covered; the absorbent core of the dressing should overlap the wound by a minimum of 2 cm. Make sure that meco® Superabsorber is in close contact with the wound bed or with the primary dressing. Secure the product in place. meco® Superabsorber may be used in combination with compression therapy. To prevent the dressing from becoming completely saturated, check it regularly in accordance with best practice in the management of exuding wounds. Moisten the dressing with saline solution to make it easier to remove. Dispose of the dressing in the normal waste. If it is contaminated, follow the local regulations for disposal.

Caution:

Do not cut the dressing: the material must remain intact. Do not use the dressing if the sterile packaging is damaged. Single-use product, do not resterilise, do not re-use.

Contraindications:

meco® Superabsorber should not be used on dry or low exuding wounds, eyes, mucous membranes, tendons or in wound cavities because the dressing swells during absorption. meco® Superabsorber should not be used on persons with a known sensitivity to the dressing or its components.

Do not cut the superabsorbent wound dressing.

Do not use if package is damaged.

Do not re-use. Re-using will cause cross-contamination.

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