# IT

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**Manufacturer:**  
Zhejiang Longterm Medical Technology Co., Ltd. Huancheng North Road 493,  
Mo Gan Mountain National High-Tech District, 313200 Deqing, Zhejiang,  
PEOPLE'S REPUBLIC OF CHINA

**EC REP**

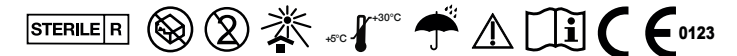
**EU Representative:**  
Lotus NL B.V.  
Koningin Julianaplein 10  
1e Verd  
2595AA The Hague  
THE NETHERLANDS

**Importer:**  
White Label Pharma GmbH  
Hohenzollernring 38-40  
50672 Köln  
GERMANY

**Distributor:**  
meco Store GmbH  
In den Auwiesen 6  
69168 Wiesloch  
GERMANY

# EN

## Product description

meco® Foam Heel Non-Adhesive is a sterile polyurethane foam dressing which effectively absorbs and retains wound exudate. Its permeable structure accelerates and improves its absorption capacity. On contact with the wound exudate, an ideally moist wound environment is created which thus promotes granulation and epithelialisation. meco® Foam Heel Non-Adhesive is soft, elastic and well tolerated.

Depending on the clinical indication, meco® Foam Heel Non-Adhesive can be secured in place with a secondary dressing or a compression bandage.

## Indications

meco® Foam Heel Non-Adhesive is suitable for a wide variety of exuding wounds, including:

- pressure sores
- diabetic foot
- post-operative wounds
- and first and second-degree burn injuries

## Contraindications

meco® Foam Heel Non-Adhesive should not be used on patients allergic to one of the components of the foam dressing. meco® Foam Heel Non-Adhesive should not be used on wounds covered with a dry scab or a firm black necrosis before this has been surgically or otherwise professionally removed.

## Method of use

1. Clean the wound in accordance with standard medical practice and dry the surrounding skin.
2. Select a dressing size which extends at least 2-3 cm beyond the wound margin. Place the dressing with the soft side on the wound and secure it in place with adhesive tape or a cohesive bandage as required. Select the appropriate dressing for areas of the body that are difficult to dress, such as the heels or sacral area.
3. meco® Foam Heel Non-Adhesive may be applied beneath a compression dressing.

Disposal should be carried out in compliance with local environmental protection requirements.

## Precautions and observations

meco® Foam Heel Non-Adhesive is for single use only and must not be reused. Reuse can be associated with an increased risk of infection or cause cross contamination. In addition, the physical properties of the product may no longer be ideal for its intended purpose. This wound dressing should not be used with any other wound care products without prior consultation with a medical specialist.

Do not use meco® Foam Heel Non-Adhesive with oxidising agents such as hydrogen peroxide or hypochlorite solutions.

If you notice any skin irritations (reddening, inflammation), maceration (softening of the skin associated with white discolouration), hypergranulation (overgrowth of tissue) or signs of infection (progressive pain, bleeding, warmth/redness of the surrounding tissue, wound exudate) or a change in the wound colour and/or smell, please consult your doctor or the nursing staff.

## Dressing change frequency

How often the dressing should be changed usually depends on the amount of wound exudate. meco® Foam Heel Non-Adhesive may remain on the wound for up to 7 days, depending on the condition of the wound and the surrounding skin or in accordance with clinical practice.

## Storage

meco® Foam Heel Non-Adhesive should be stored in a clean, well-ventilated, darkened space with a relative humidity below 80%, a room temperature of +5°C to +30°C, and free of corrosive gases.

**STERILE R** Sterilization by radiation

Do not reuse

Do not use the product if package is damaged.

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